


Restock Prescription Labels in DOSIS


1. Use the printer drawer handle and fully pull the printer out from the cabinet.
2. Remove the cardboard core from the plastic roll guide and keep it for later use.

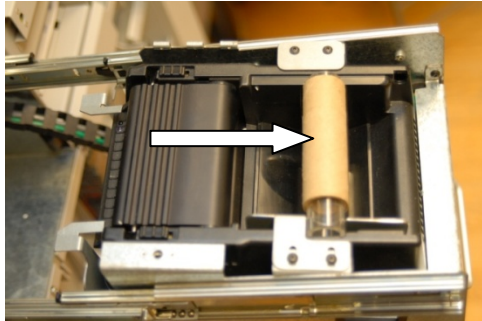

3. Remove the take-up spool from under the printer by pulling the retaining knob off and pulling on the spent leader and core. Throw them away.




4. Open the print head by pushing the release latches towards the back of the printer.

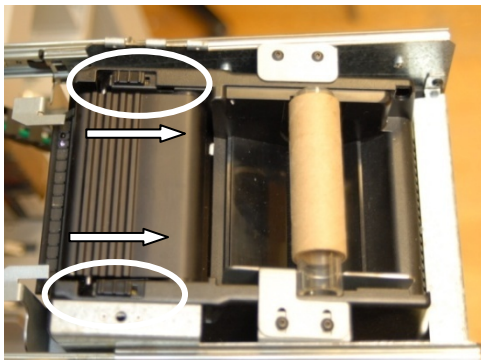


5. Put the plastic label roll guide through the core of the labels and place the roll into the printer


with the leader unwinding from the bottom of the roll towards the inside of the cabinet.

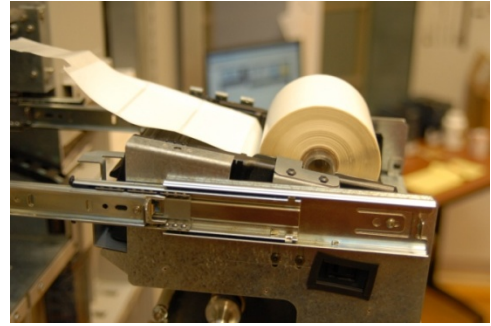

6. Remove 6-8 labels from the leader and throw them away.
7. Feed the media under the print head.

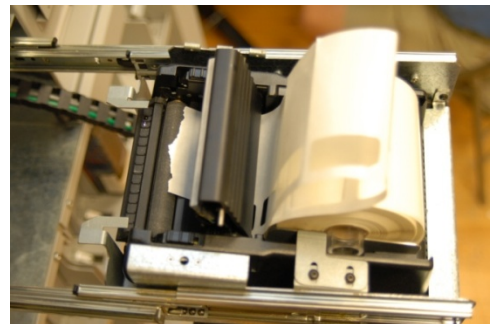

8. Feed the leader through the peeler by lifting the peeler up to expose the slot. Continue holding the peeler up as you feed the leader through it.

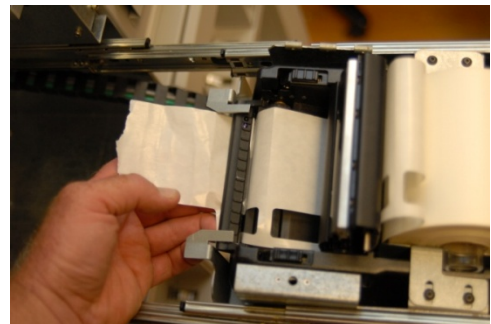


9. Place a label on the end of the leader and secure it to the old core from Step 2.



In-Cabinet Prescription Label Restocking Guide

10. Ensure the leader is under the tension bar and push the core onto the take up reel and put the retaining knob back in place.

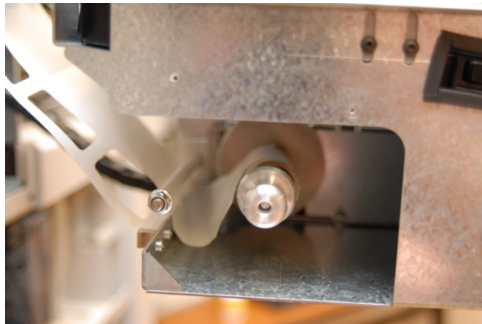


11. Press the take-up spool switch until the media leader is tight and the take-up spool quits turning.




12. Test the printer by pressing the Feed button on the front of the printer.




13. The label should feed out and peel.

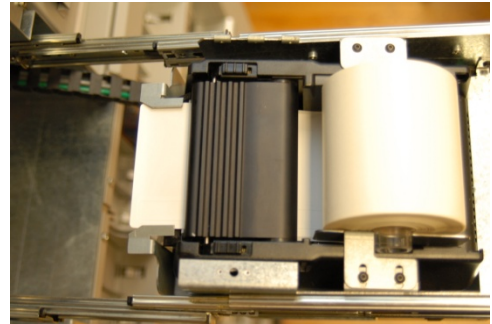

14. Remove the label and tighten the tensioner.

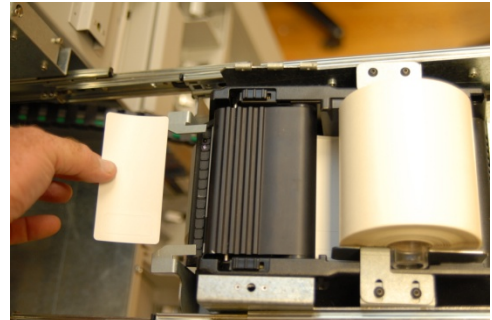

15. Push the printer back into the machine ensuring the drawer handle is flat against the drawer.

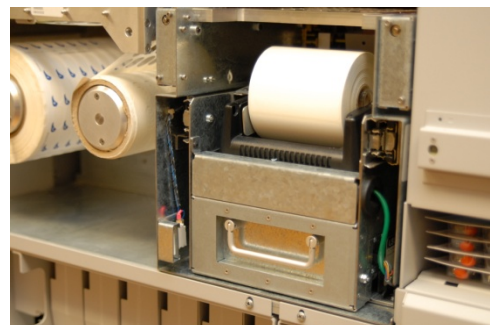

16. Close the access door and press the Run button on the GUI to continue filling prescriptions.